

ECLIPSE MICROPEN™


Installing Needle Tip

Install the needle tip by pressing tip down and rotating clockwise until the tip will turn no further. Remove the tip by rotating the tip counter clockwise until it releases.



Adjusting Needle Depth

The needle depth ranges from a depth of 0.0mm to 2.75mm by turning the dial left (deeper), right (shallower). The arrow points to the current depth.

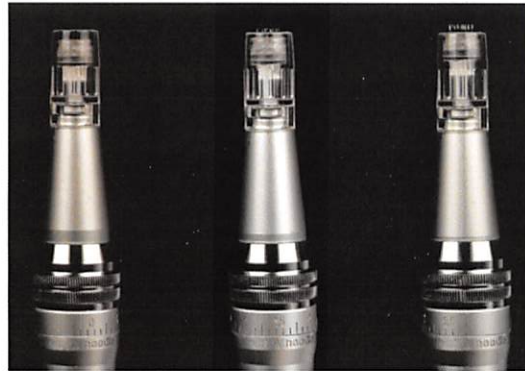

Charging

Screw the battery unit into the charging unit. Insert the charger adapter into the charging unit and insert the plug into a power outlet. Press the On/Off button on the battery until the charger adapter turns red. Charge until the adapter turns green (approx. 75 minutes). After charging, unscrew the battery and screw into the MicroPen™. On/Off button is on the bottom. Running time is approximately 4 hours.

